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dyras

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Modell: BTPS-2401B

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Model: BTPS-2401B

with „OKOK International” app

Before commissioning this device please read the instruction manual carefully.

CAUTION

Please read the user manual thoroughly before using your new appliance. It is important to keep these instructions as a useful reminder that will help you enjoy the many features available. The manufacturer is not responsible if the appliance becomes damaged or defective as a result of the user not following the information included in the instruction manual. If the appliance becomes faulty during regular use, it should only be repaired by an authorized service engineer.

IMPORTANT SAFETY INFORMATION

You must obey the safety instructions that follow when using the appliance. Please read this section very carefully!

Of course, it is not possible to create safety regulations for every life situation, which is why we ask you to take all reasonable precautions in any given situation, even when you are taking the utmost care, in order to ensure safe operating conditions.

- This device is not supposed to be used by physically or mentally handicapped people, or persons without experience or knowledge (including children), unless their use of the device is supervised or guided by someone who is at the same time responsible for their safety. As for children, supervision is recommended to make sure that they do not play with the device. This device is not supposed to be used by children under the age of 14, not even with supervision!
- Never use an appliance or accessory that the manufacturer did not supply with, or propose for use with, the appliance. The use of such appliances or accessories might lead to accidents or irregular function.
- Do not place the appliance near hot objects (e.g. electric or gas cooker, hob, radiator, etc.) or on a hot surface and do not use the appliance on an unstable surface, or a high surface, or any surface where it is unstable and can be knocked down in an accident.
- Do not use the appliance in the presence of explosive or inflammable gases, vapours or liquids.

“OKOK International” APP

You can access all the functions of the scale using the “OKOK International” application.

Supported operation system:

- Android 4.3 and above for selected Android devices with Bluetooth 4.0 and above.
- IOS

Install the app which you can easily download from the App Store or Google Play.

Connect the unit with your mobile or any other device via Bluetooth.

The analysis will be shown in the app.

The app can save data for up to 8 people, so you and your family and friends can use this scale.

Download now:
OKOK International

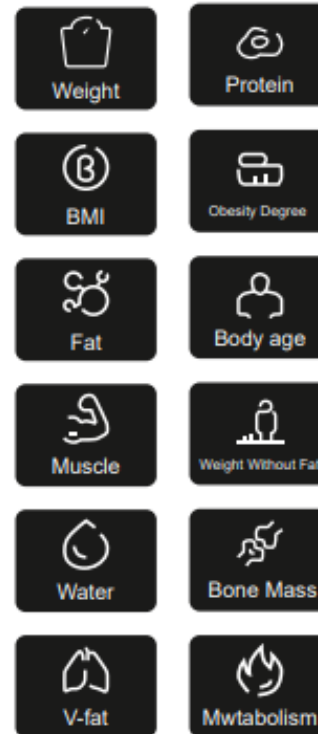
ANDROID APP ON
Google play

Available for iPhone
App Store

OKOK Healthcare

THE MENU

History (previous data)

Dynamic (present data)

Trend (diagrams)

Device (your appliance)

Me (user profile)

HOW TO USE THE SCALE

The device needs to be connected with the app before first time use. The weight scale will automatically connect to your phone and the application.

Make sure the Bluetooth is activated on your phone before you start using the weight scale.

1. Switch on Bluetooth. Scan the QR Code by your phone or search “OKOK International” app in App Store/ Google Play. Install the app.
2. Pair the device, register (insert user information: name, gender, height, birthday, target weight and choose a profile picture). Click next to confirm your personal information.
3. Weigh and synchronize: Step on the scale and weight yourself. Please wait till the device calculates your measurements.
4. View the result.

The device will display the following data:

Weight (kg)

BMI

Fat (%)

Muscle (kg)

Water (%)

Visceral fat,

Bone mass (kg)

Metabolism

Protein (%)

Obesity (%)

Body age (year)

LBM (kg)

The device will save your details automatically.

Open the app and connect to Bluetooth for all new weighing. Pair the device.

Function:

- The user can type weight data into the app to keep a track.
- The device will display current weight and generate reports and give health advise based on measurements. Including current weight data, weight list, data report and notices.
- Weight trend will include weekly report, monthly report and quarterly report, indicating changes of weight through diagrams.

- Account management includes registration, login, changing password, handling forgotten password, links to social media and more.
- User management includes user information like nickname, birthday, height, gender, etc.

INSTRUCTIONS FOR USE

- Remove the insulation film from the back of the scale near the battery compartment prior to operation.
- Place the batteries in the scale. Pay attention to the correct direction (“+” and “-“).
- Press the button on the bottom of the scale, near the battery box to choose the different unit: kg/lb.
- Place the scale on a plane and hard surface prior to weighting.
- Pair your device with scale.
- Step on the scale, balance your weight evenly and wait until the display reads your weight. Your weight will flash on the display for 10 seconds.
- The weight appears automatically in the app.
- Change batteries if you see “Lo” sign on the display.

CAUTION

- Please do not leave any objects on the surface of the scale if you do not use it.
- Handle the scale with care, as it is a precise measuring instrument. Do not drop it or jump on it.
- Do not load the appliance with more than 180 kg to avoid damaging the device.
- Do not allow your scale to be wet, it may damage the internal parts. Never immerse it into water or other liquid.
- If the scale is not going to be used, please store it only in a horizontal position.
- Do not get on it with wet feet, you could fall down
- The body fat analysis scale is not suitable for the following persons:

- Children under 10 years and adults over 99 years
- Persons with symptoms of fever, edema, or osteoporosis
- Persons undergoing treatment for dialysis
- Persons who use a pacemaker
- Persons who take cardiovascular medication
- Pregnant women
- Sportspersons who perform over 5 hours of intensive sporting activities each day
- Persons with a heartbeat lower than 60 beats/minute

ATTENTION

- Do not leave any object on the scale when not in use.
- Keep the scale surface and your feet drying when weighting.
- Make sure you stand in the center of the scale to avoid tipping.
- Avoid dropping or hitting the scale.
- Wipe scale surface with a clean dry cloth if need clean.
- Do not wash the scale with water, the scales mechanism can be damaged if immersed in water.

HOW TO CLEAN

- Wipe scale surface with a clean dry cloth if need clean.
- Do not wash the scale with water, the scale's mechanism can be damaged if immersed in water.

SPECIFICATIONS

Model: BTPS-2401B

- Power: 3XAAA battery
- Capacity: 180kg/396lb
- Unit: kg/lb
- Memory for 8 persons
- Measure weight: BMI/fat/V-fat/water/muscle/bone/Calorie
- Turn on auto
- Auto off in 10 seconds
- Low battery display: "Lo"
- Overweight display: "oL"

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THE CE SYMBOL

The CE mark is intended to indicate that the product complies with the relevant European Union standards and can be freely marketed in the European Union.

Correct Disposal of this product	
	<p>This marking indicates that this product should not be disposed with other household wastes throughout the EU. To prevent possible harm to the environment or human health from uncontrolled waste disposal, recycle it responsibly to promote the sustainable reuse of material resources. To return your used device, please use the return and collection systems or contact the retailer where the product was purchased. They can take this product for environmental safe recycling.</p>

dyras

www.dyras.hu

Copyright © 2022 The dyras wwe.factory LLC U.S.A.

No parts of the content (including logos, design, photos, text, colour scheme arrangements, etc.) may be reproduced, resued or redistributed for any purpose whatsoever, or distributed to a thirdparty for such purposes, without the written permission of The dyras wwe.factory LLC.

®= registered trademark of Dyras Europe Kft.